

Elim COVID Saliva Test

This Fact Sheet informs you of the significant known and potential risks and benefits of Elim COVID Saliva Test (Elim COVID assay). The Elim COVID assay utilizes saliva specimens collected from individuals. Elim COVID is a Laboratory Developed Test (LDT).

What are the symptoms of COVID-19?

Many patients with COVID-19 have developed fever and/or symptoms of acute respiratory illness (e.g., cough, dyspnea), although some individuals experience only mild symptoms or no symptoms at all. The current information available to characterize the spectrum of clinical illness associated with COVID-19 suggests that, when present, symptoms include cough, shortness of breath or dyspnea, fever, chills, myalgias, headache, sore throat, new loss of taste or smell, nausea or vomiting or diarrhea. Signs and symptoms may appear any time from 2 to 14 days after exposure to the virus, and the median time to symptom onset is approximately 5 days. For further information on the symptoms of COVID-19 please see the link provided in “*Where can I go for updates and more information?*” section.

Public health officials have identified cases of COVID-19 infection throughout the world, including the United States. Please check the CDC COVID-19 webpage (see link provided in “*Where can I go for updates and more information?*” section at the end of this document) or your local jurisdictions website for the most up to date information.

What do I need to know about COVID-19 testing?

Current information on COVID-19 for healthcare providers is available at CDC’s webpage, *Information for Healthcare Professionals* (see links provided in “*Where can I go for updates and more information?*” section).

This test is to be performed only using saliva specimens collected from individuals:

- The Elim COVID assay can be used to test saliva specimens.
- The Elim COVID assay should be ordered for the detection of COVID-19 in any individual
- The assay is authorized for use for the Elim Clinical Laboratory, which is certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA), 42 U.S.C. §263a and meets the requirements to perform high complexity tests.

Specimens should be collected with appropriate infection control precautions. Current guidance is available at the CDC’s website (see links provided in “*Where can I go for updates and more information?*” section).

When collecting and handling specimens from individuals suspected of being infected with COVID-19, appropriate personal protective equipment should be used as outlined in the CDC *Interim Laboratory Biosafety Guidelines for Handling and Processing Specimens Associated with Coronavirus Disease 2019 (COVID-19)*. For additional information, refer to CDC *Interim Guidelines for Collecting, Handling, and Testing Clinical Specimens from Persons Under Investigation (PUIs) for Coronavirus Disease 2019 (COVID-19)* (see links provided in “*Where can I go for updates and more information?*” section).

What does it mean if the specimen tests positive for the virus that causes COVID-19?

A positive test result for COVID-19 indicates that RNA from SARS-CoV-2 was detected, and therefore the patient is infected with the virus and presumed to be contagious. Laboratory test results should always be considered in the context of clinical observations and epidemiological data (such as local prevalence rates and current outbreak/epicenter locations) in making a final diagnosis and patient management decisions. Patient management should be made by a healthcare provider and follow current CDC guidelines.

The Elim COVID assay has been designed to minimize the likelihood of false positive test results. However, it is still possible that this test can give a false positive result, even when used in locations where the prevalence is below 5%. In the event of a false positive result, risks to patients could include the following: a recommendation for isolation of the patient, monitoring of household or other close contacts for symptoms, patient isolation that might limit contact with family or friends and may increase contact with other potentially COVID-19 patients, limits in the ability to work, delayed diagnosis and treatment for the true infection causing the symptoms, unnecessary prescription of a treatment or therapy, or other unintended adverse effects.

All laboratories using this test must follow the standard testing and reporting guidelines according to their appropriate public health authorities.

What does it mean if the specimen tests negative for the virus that causes COVID-19?

A negative test result for this test means that SARS- CoV-2 RNA was not present in the specimen above the limit of detection. However, a negative result does not rule out COVID-19 and should not be used as the sole basis for treatment or patient management decisions. It is possible to test a person too early or too late during COVID-19 infection to make an accurate diagnosis via the Elim COVID assay.

When diagnostic testing is negative, the possibility of a false negative result should be considered in the context of a patient's recent exposures and the presence of clinical signs and symptoms consistent with COVID-19. The possibility of a false negative result should especially be considered if the patient's recent exposures or clinical presentation indicate that COVID- 19 is likely, and diagnostic tests for other causes of illness (e.g., other respiratory illness) are negative. If COVID-19 is still suspected based on exposure history together with other clinical findings, re-testing with an alternative method should be considered by healthcare providers in consultation with public health authorities. Additional testing may be helpful to ensure testing was not conducted too early.

If a negative result is obtained with a saliva specimen and COVID-19 is still suspected based on exposure history together with other clinical findings, testing an alternative specimen type should be considered by healthcare providers in consultation with public health authorities.

Risks to a patient of a false negative test result include: delayed or lack of supportive treatment, lack of monitoring of infected individuals and their household or other close contacts for symptoms resulting in increased risk of spread of COVID-19 within the community, or other unintended adverse events.

Where can I go for updates and more information?

CDC webpages:

General: <https://www.cdc.gov/COVID19>

Symptoms: <https://www.cdc.gov/coronavirus/2019-ncov/symptoms- testing/symptoms.html>

Healthcare Professionals: <https://www.cdc.gov/coronavirus/2019-nCoV/guidance- hcp.html>

Information for Laboratories: <https://www.cdc.gov/coronavirus/2019-nCoV/guidance- laboratories.html>

Laboratory Biosafety: <https://www.cdc.gov/coronavirus/2019-nCoV/lab-biosafety- guidelines.html>

Isolation Precautions in Healthcare Settings: <https://www.cdc.gov/coronavirus/2019-ncov/infection- control/control-recommendations.html>

Specimen Collection: <https://www.cdc.gov/coronavirus/2019-nCoV/guidelines- clinical-specimens.html>

Infection Control: <https://www.cdc.gov/coronavirus/2019- ncov/infection-control/index.html>

FDA webpages:

General: www.fda.gov/novelcoronavirus